


SnapIT


Multi-Use | Ampoule Opener

施耐柏安瓿安全開瓶器

【產品使用說明】

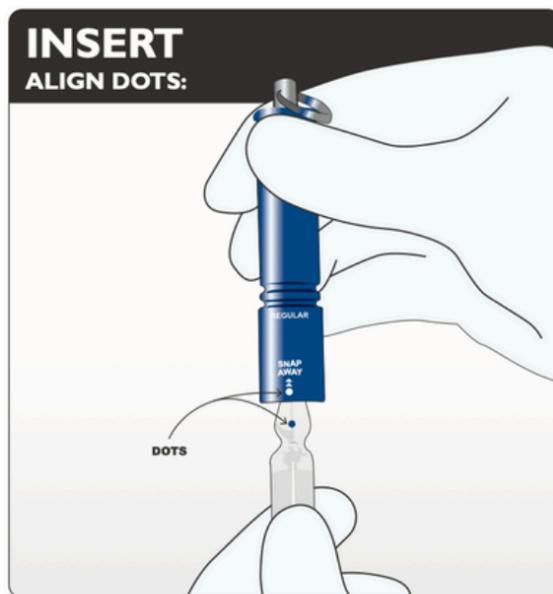

1. 安瓿上若有標示點則務必對準SnapIT上的標示點，輕緩地將安瓿上部套入SnapIT，使其固定於頸部的位罝。
2. 將標示點朝外(不要面向自己)，與標示點反方向輕輕施力折斷安瓿的上部且其會保留在SnapIT內。
3. 以大拇指按壓SnapIT上方的樞紐中軸，將安瓿上部壓出棄置於尖銳物收集桶內。

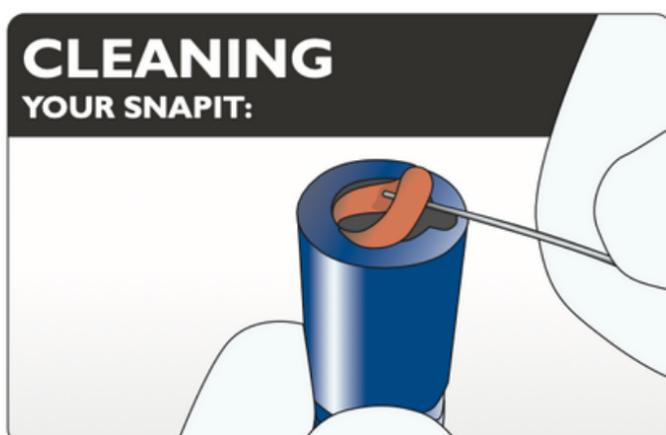

4. 定期清潔: 請依圖示取出O型環，再以清水或醫院用等級清潔劑清洗之。請在清潔盤或容器內進行此步驟以避免零件不見。待完全乾燥後重新組裝即可再使用。

Notes :

如欲將SnapIT全部拆解清洗，請先將SnapIT上方套環拆下，O型環及中軸取出，在按照上方步驟4完成清潔動作。O型環為可替換零件，若有需要購買請洽客服。

施耐柏安瓿安全開瓶器 ~LITE 輕量系列 ~

SnapIT 保護您的雙手, 更簡單、安全、快速的開啟安瓿, 避免因徒手開啟造成尖銳物刺傷, 降低工作職業傷害。可重覆使用, CP值高。



【產品規格】

- 產品系列：SnapIT Lite (輕量系列) 適合個人隨身攜帶使用
- 產品型號：SN-02R，SN-02L
- 產品材質：本體ABS 塑鋼; O型環 Silicon rubber (O型環可替換)
- 產品尺寸：Regular ; Large
- 產品顏色：鐵灰色

【如何挑選適合的SnapIT】

- 請挑選能套入且固定卡住安瓿的SnapIT規格。(Regular或Large)
- SnapIT選擇決定於安瓿瓶頸的大小, ml量僅為參考

Sizes

- If the ampoule lid is too big, it will not fit through the entrance. Use a larger Snapit.
- If the ampoule is too small, the Snapit will not hold the top securely. Use a smaller Snapit.

Ampoule Size	Snapit Size	Ampoule Size	Snapit Size
1 - 2 ml	REGULAR Most Common Sizes	Large 5 - 10 ml	LARGE
5 - 10 ml		10 - 15 ml	
10 - 15 ml		20 - 25 ml	

Snapit size is determined by Ampoule lid. Content in mls can be used as a guide.

Due to the inconsistent variation of tops, this is only a guideline.

【注意事項】

- LITE輕量系列不適用高溫高壓消毒,如有高溫高壓消毒需求, 請洽客服詢問 Personal 與 Trolley 系列 鋁合金材質。
- Regular+Large合用可開啟各種尺寸的安瓿瓶。(只有極少數的安瓿瓶頭為圓形 或過小無法卡入開瓶器中。)